

Directions of use:

Note: NuMedical Carbide Burs and Diamond Burs are non-sterile and are manufactured for dental use only.

1. The device is to be used on the instruction of, or by a dentist or other licensed practitioner.
2. Clean and sterilise non-sterile burs in accordance with the validated procedure prior to first use and prior to each reuse.
3. Do not force bur into the handpiece. In case of difficult access, check both handpiece turbine and bur and refer to handpiece instructions for troubleshooting.
4. Do not use worn-out or dull burs.
5. Do not apply excessive pressure on the bur as this could cause undesirable heat or may cause the bur to fail.
6. Avoid removing the bur at too sharp an angle to avoid leverage and breakage.
7. Carefully read package labels to ensure use of the appropriate device.
8. Move the bur continuously when in use to avoid localised heating and/or damage to the bur.
9. Maintain handpieces in good working condition to ensure maximum effectiveness of the device.
10. Ensure the bur is fully seated and securely gripped in the handpiece collet prior to use.
11. Always wear gloves when handling contaminated instruments. **Refer to page 141 for gloves.**
12. Eye protection must be worn to protect against eject particles. **Refer to page 170 - 172 for eye wear.**
13. Surgical masks must be worn to avoid inhalation of any aerosol or dust generated by the device. **Refer to page 1 - 10 for masks.**

Carbide Bur - Pear 5pcs/pack

TYPE	Price
FG	\$11.65/pack
FG XL	\$19.65/pack
RA	\$12.85/pack

ISO	Size #	Head Size	Head Length	Total Length	FG Code	FG XL Code	RA Code	Actual Length						
								0	5	10	15	20	25	
500 314 237 006 008	330	0.8	1.9	19	994860									
500 314 237 006 008	330	0.8	1.9	19		994861								
500 204 237 006 012	330	0.8	1.9	22.5			994862							
500 314 237 006 010	331	1.0	2.0	19	994863									
500 204 237 006 010	331	1.0	2.0	22.5			994864							
500 314 237 006 012	332	1.2	2.2	19	994865									
500 204 237 006 012	332	1.2	2.2	22.5			994866							
500 314 139 008 009	245	0.9	2.5	19	994867									

Carbide Bur - Straight Flat End 5pcs/pack

TYPE	Price
FG	\$11.65/pack
FG XL	\$19.65/pack
RA	\$12.85/pack

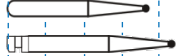
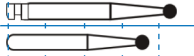
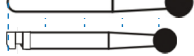
ISO	Size #	Head Size	Head Length	Total Length	FG Code	FG XL Code	RA Code	Actual Length						
								0	5	10	15	20	25	
500 314 107 007 008	556	0.8	3.8	19	994868									
500 204 107 007 008	556	0.8	3.8	22.5			994869							
500 314 107 007 010	557	1.0	4.2	19	994870									
500 316 107 007 101	557	1.0	4.2	25		994871								
500 204 107 007 010	557	1.0	4.2	22.5			994872							
500 314 107 007 012	558	1.2	4.2	19	994873									
500 316 107 007 012	558	1.2	4.2	25		994874								
500 204 107 007 012	558	1.2	4.2	22.5			994875							

BURS

Carbide Bur - Round

5pcs/pack

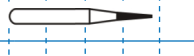
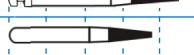
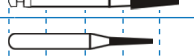
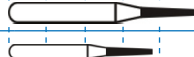
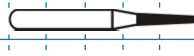
TYPE	Price
FG	\$11.65/pack
FG XL	\$19.65/pack
RA	\$12.85/pack

ISO	Size #	Head Size	Head Length	Total Length	FG Code	FG XL Code	RA Code	Actual Length						
								0	5	10	15	20	25	
500 314 001 001 008	1	0.8	0.8	19	994821									
500 204 001 001 008	1	0.8	0.8	22.5			994822							
500 314 001 001 010	2	1.0	1.0	19	994823									
500 316 001 001 010	2	1.0	1.0	25		994824								
500 204 001 001 010	2	1.0	1.0	22.5			994825							
500 314 001 001 012	3	1.2	1.2	19	994826									
500 316 001 001 012	3	1.2	1.2	25		994827								
500 204 001 001 012	3	1.2	1.2	22.5			994828							
500 314 001 001 014	4	1.4	1.4	19	994829									
500 316 001 001 014	4	1.4	1.4	25		994830								
500 204 001 001 014	4	1.4	1.4	22.5			994831							
500 314 001 001 016	5	1.6	1.6	19	994832									
500 204 001 001 016	5	1.6	1.6	22.5			994833							
500 314 001 001 018	6	1.8	1.8	19	994834									
500 316 001 001 018	6	1.8	1.8	25		994835								
500 204 001 001 018	6	1.8	1.8	22.5			994836							
500 314 001 001 021	7	2.1	2.1	19	994837									
500 204 001 001 021	7	2.1	2.1	22.5			994838							
500 314 001 001 023	8	2.3	2.3	19	994839									
500 316 001 001 023	8	2.3	2.3	25		994840								
500 204 001 001 023	8	2.3	2.3	22.5			994841							

Carbide Bur - Taper Fissure

5pcs/pack

TYPE	Price
FG	\$11.65/pack
FG XL	\$19.65/pack
RA	\$12.85/pack

ISO	Size #	Head Size	Head Length	Total Length	FG Code	FG XL Code	RA Code	Actual Length						
								0	5	10	15	20	25	
500 314 168 007 008	698	0.8	3.8	19	994842									
500 314 168 007 009	699	0.9	4.0	19	994843									
500 314 168 007 010	700	1.0	4.2	19	994844									
500 204 168 007 010	700	1.0	4.2	19			994845							
500 314 168 007 012	701	1.2	4.2	19	994846									
500 316 168 007 012	701	1.2	4.2	25		994847								
500 204 168 007 012	701	1.2	4.2	19			994848							
500 314 168 007 016	702	1.6	4.5	19	994849									
500 316 168 007 016	702	1.6	4.5	25		994850								
500 204 168 007 016	702	1.6	4.5	22.5			994851							
500 314 168 007 021	703	2.1	5.0	19	994852									
500 204 168 007 021	703	2.1	5.0	22.5			994853							
500 314 168 007 023	704	2.3	5.0	19	994854									
500 204 168 007 023	704	2.3	5.0	22.5			994855							
500 314 168 007 010	701L	1.0	6.0	19	994856									
500 316 168 007 010	701L	1.0	6.0	25		994857								
500 314 168 007 012	702L	1.2	6.0	19	994858									
500 316 168 007 012	702L	1.2	6.0	25		994859								

BURS

BURS

Carbide Bur - Straight Dome End 5pcs/pack

\$12.85
/pack

ISO	Size #	Head Size	Head Length	Total Length	FG Code	RA Code	Actual Length					
							0	5	10	15	20	25
500 314 139 008 010	1957	1.0	4.2	19	994876							
500 314 138 008 010	1931	1.0	2.0	19	994877							
500 314 139 008 012	1958	1.2	4.2	19	994878							
500 314 137 007 009	1556	0.9	4.0	19	994879							
500 314 137 007 010	1557	1.0	4.2	19	994880							
500 314 137 007 012	1558	1.2	4.2	19	994881							
500 204 137 007 010	1558	1.2	4.2	22.5		994882						



Carbide Bur - Zekrya 5pcs/pack

\$27.55
/pack

ISO	Size #	Head Size	Head Length	Total Length	FG L Code	FG XL Code	FG XXL Code	Actual Length						
								0	5	10	15	20	25	30
500 315 199 295 016	151	1.4	11	23	995510									
500 316 199 295 016	151	1.4	11	25		995511								
500 204 199 295 016	151	1.4	11	28			995512							



Carbide Bur - Endo Z 5pcs/pack

\$27.55
/pack

ISO	Size #	Head Size	Head Length	Total Length	RA Code	FG L Code	FG XL Code	FG XXL Code	Actual Length							
									0	5	10	15	20	25	30	
500 204 237 006 012	152	1.2	2.2	22.5		994886										
500 316 219 295 014	152	1.4	9	25			994887									
500 316 219 295 016	152	1.6	9	28				994888								
500 204 219 295 016	152	1.6	9.0	24.5	994889											



Carbide Bur HP - Straight Dome End 5pcs/pack

\$14.25
/pack

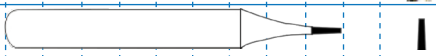
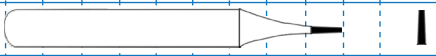
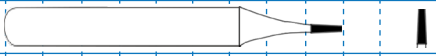
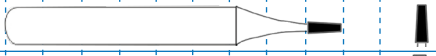
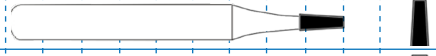
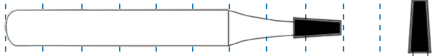
ISO	Size #	Head Size	Head Length	Total Length	HP Code	Actual Length										
						0	5	10	15	20	25	30	35	40	45	50
500 104 107 007 008	556	0.8	3.8	44.5	994905											
500 104 107 007 010	557	1.0	4.2	44.5	994906											
500 104 107 007 012	558	1.2	4.2	44.5	994907											
500 104 107 007 014	559	1.4	4.4	44.5	994908											
500 104 107 007 016	560	1.6	4.4	44.5	994909											
500 104 107 007 018	561	1.8	4.9	44.5	994910											
500 104 107 007 021	562	2.1	5.1	44.5	994911											
500 104 107 007 023	563	2.3	5.4	44.5	994912											



BURS

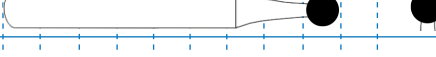
Carbide Bur HP - Taper Flat End
(Cross Cut)
5pcs/pack

\$14.25
/pack

ISO	Size #	Head Size	Head Length	Total Length	HP Code	Actual Length	
500 104 168 007 009	699	0.9	3.8	44.5	994899	0 5 10 15 20 25 30 35 40 45 50	
500 104 168 007 010	700	1.0	4.2	44.5	994900		
500 104 168 007 012	701	1.2	4.2	44.5	994901		
500 104 168 007 016	702	1.6	4.4	44.5	994902		
500 104 168 007 021	703	2.1	4.9	44.5	994903		
500 104 168 007 023	704	2.3	5.4	44.5	994904		

Carbide Bur HP - Round
5pcs/pack

\$14.25
/pack

ISO	Size #	Head Size	Head Length	Total Length	HP Code	Actual Length	
500 104 001 001 006	1/2	0.6	0.6	44.5	994890	0 5 10 15 20 25 30 35 40 45 50	
500 104 001 001 008	1	0.8	0.8	44.5	994891		
500 104 001 001 010	2	1.0	1.0	44.5	994892		
500 104 001 001 012	3	1.2	1.2	44.5	994893		
500 104 001 001 014	4	1.4	1.4	44.5	994894		
500 104 001 001 016	5	1.6	1.6	44.5	994895		
500 104 001 001 018	6	1.8	1.8	44.5	994896		
500 104 001 001 021	7	2.1	2.1	44.5	994897		
500 104 001 001 023	8	2.3	2.3	44.5	994898		

Carbide Bur HP - Round L
5pcs/pack

\$29.95
/pack

ISO	Size #	Head Size	Head Length	Total Length	HP Code	Actual Length	
500 104 001 001 006	2	0.6	0.6	65	994913	0 5 10 15 20 25 30 35 40 45 50 55 60 65 70	
500 104 001 001 008	3	0.8	0.8	65	994914		
500 104 001 001 010	4	1.0	1.0	65	994915		
500 104 001 001 012	5	1.2	1.2	65	994916		
500 104 001 001 014	6	1.4	1.4	65	994917		
500 104 001 001 016	7	1.6	1.6	65	994918		
500 104 001 001 018	8	1.8	1.8	65	994919		

BURS

Nu
medical

BURS

Diamond Bur - Taper Flat End
10pcs/pack

\$9.65
/pack

Coarse / 534 / 125-150 μ

Standard / 524 / 106-125 μ

Fine / 514 / 53-63 μ

ExtraFine / 504 / 20-30 μ



ISO	PRODUCT CODE			SIZE & LENGTH			ACTUAL LENGTH					
				Head Diameter	Head Length	Total Length	0	5	10	15	20	25
Diamond, Type, Shape												
806 314 169	TF-S41	994619	Standard	1.1	4	16.5						
806 314 169	TF-41	994495	Standard	1.1	4	19						
806 314 170	TF-42	994496	Standard	1.3	4	19						
806 314 170	TF-43	994497	Standard	1.4	4.5	19						
806 314 170	TF-SS31	994660	Standard	1.6	4.5	14.5						
806 314 170	TF-S31	994618	Standard	1.6	4.5	17						
806 314 170	TF-31	994494	Standard	1.6	4.5	19						
806 314 170	TF-31C	994682	Coarse	1.6	4.5	19						
806 314 170	TF-45C	994683	Coarse	2.3	4.5	20						
806 314 170	TF-S23	994616	Standard	1.8	5	17						
806 314 170	TF-23	994492	Standard	1.8	5	19						
806 314 170	TF-S22	994615	Standard	2.1	5.5	17						
806 314 170	TF-22	994491	Standard	2.1	5.5	20						
806 314 171	TF-S21	994614	Standard	1.6	7	19						
806 314 171	TF-21	994490	Standard	1.6	7	21						
806 314 171	TF-21F	994730	Fine	1.6	7	21						
806 314 171	TF-21EF	994776	ExtraFine	1.6	7	21						
806 314 171	TF-S20	994613	Standard	1.4	7	19						
806 314 171	TF-20	994489	Standard	1.4	7	21						
806 314 198	TF-S24	994617	Standard	1.2	8	18						
806 314 198	TF-24	994493	Standard	1.2	8	20						
806 314 172	TF-S18	994611	Standard	1.2	8	18						
806 314 172	TF-18	994487	Standard	1.6	8	20						
806 314 172	TF-S19	994612	Standard	1.3	8	18.5						
806 314 172	TF-19	994488	Standard	1.3	8	20						
806 314 172	TF-19C	994681	Coarse	2.3	8	20						
806 314 172	TF-14	994483	Standard	2.3	10	22						
806 314 172	TF-L14	994666	Standard	2.3	10	24						
806 314 173	TF-S11	994608	Standard	1.4	10	22						
806 314 173	TF-11	994480	Standard	1.4	10	24						
806 314 173	TF-11C	994676	Coarse	1.4	10	24						
806 314 173	TF-S12	994609	Standard	1.6	10	20						
806 314 173	TF-12	994481	Standard	1.6	10	22						
806 314 173	TF-12C	994677	Coarse	1.6	10	22						
806 314 173	TF-12F	994729	Fine	1.6	10	22						
806 314 173	TF-12EF	994775	ExtraFine	1.6	10	22						
806 314 173	TF-L12	994665	Standard	1.6	10	24						
806 314 173	TF-17	994486	Standard	1.8	10	22.5						
806 314 173	TF-15	994484	Standard	1.9	10	22.5						
806 314 173	TF-15C	994679	Coarse	1.9	10	22.5						
806 314 173	TF-16	994485	Standard	1.7	10.3	22						
806 314 173	TF-16C	994680	Coarse	1.7	10	22						
806 314 173	TF-S13	994610	Standard	1.8	10.5	20						
806 314 173	TF-13	994482	Standard	1.8	10.5	22						
806 314 173	TF-13C	994678	Coarse	1.8	10.5	22						

BURS

NuMedical

BURS

Diamond Bur - Taper Round End
10pcs/pack

\$9.65
/pack

Coarse / 534 / 125-150 μ

Standard / 524 / 106-125 μ

Fine / 514 / 53-63 μ

ExtraFine / 504 / 20-30 μ



BURS

NuMedical

ISO	PRODUCT CODE			SIZE & LENGTH			ACTUAL LENGTH					
				Head Diameter	Head Length	Total Length	0	5	10	15	20	25
806 314 197	TR-62C	994689	Coarse	1.8	7	20						
806 314 197	TR-20	994505	Standard	1.4	7	21						
806 314 197	TR-SS21	994661	Standard	1.2	7	18						
806 314 197	TR-S21	994623	Standard	1.6	7	19						
806 314 197	TR-21	994506	Standard	1.6	7	21						
806 314 197	TR-21F	994733	Fine	1.6	7	21						
806 314 197	TR-21EF	994779	ExtraFine	1.6	7	21						
806 314 197	TR-24	994509	Standard	1.8	7	21						
806 314 198	TR-S23	994625	Standard	1.2	7.5	17						
806 314 198	TR-23	994508	Standard	1.2	7.5	19						
806 314 198	TR-S22	994624	Standard	1.1	8	18						
806 314 198	TR-22	994507	Standard	1.1	8	20						
806 314 198	TR-16	994503	Standard	1.8	8	21						
806 314 198	TR-S27	994626	Standard	1.3	8	21						
806 314 198	TR-27	994512	Standard	1.3	8	21						
806 314 199	TR-26	994511	Standard	1.8	8.5	21.5						
806 314 199	TR-26C	994688	Coarse	1.8	8.5	21.5						
806 314 199	TR-26F	994735	Fine	1.8	8.5	21.5						
806 314 199	TR-26EF	994781	ExtraFine	1.8	8.5	21.5						
806 314 198	TR-S29	994627	Standard	1.4	9	22						
806 314 198	TR-29	994513	Standard	1.4	9	22						
806 314 198	TR-S14C	994727	Coarse	2.2	9.0	19.5						
806 314 198	TR-14C	994687	L.Coarse	2.2	9.0	21.5						
806 314 198	TR-14	994501	Standard	2.2	9.5	22						
806 314 198	TR-L14	994668	Standard	2.2	10	24						
806 314 198	TR-S13	994622	Standard	1.8	9.5	21						
806 314 198	TR-13	994500	Standard	1.8	9.5	22						
806 314 198	TR-13C	994686	Coarse	1.8	10.5	22						
806 314 198	TR-13F	994732	Fine	1.8	9.5	22						
806 314 198	TR-13EF	994778	ExtraFine	1.8	9.5	22						
806 314 199	TR-S11	994620	Standard	1.6	10	20						
806 314 199	TR-11	994498	Standard	1.6	10	22						
806 314 199	TR-11C	994684	Coarse	1.6	10	22						
806 314 199	TR-11F	994731	Fine	1.6	10	22						
806 314 199	TR-11EF	994777	ExtraFine	1.6	10	22						
806 314 199	TR-S12	994621	Standard	1.6	10	20						
806 314 199	TR-12C	994685	Coarse	1.6	10	22						
806 314 199	TR-12	994499	Standard	1.6	10	22						
806 314 199	TR-L12	994667	Standard	1.6	10	24						
806 314 199	TR-25	994510	Standard	1.6	10	21.5						
806 314 199	TR-25F	994734	Coarse	1.6	10	21.5						
806 314 199	TR-25EF	994780	ExtraFine	1.6	10	21.5						
806 314 200	TR-15	994502	Standard	2.2	11.5	24						
806 314 200	TR-19	994504	Standard	2.4	11.5	23.5						

BURS

Diamond Bur - Chamfur
10pcs/pack

\$9.65
/pack

Coarse / 534 / 125-150 μ

Standard / 524 / 106-125 μ

Fine / 514 / 53-63 μ

ExtraFine / 504 / 20-30 μ



ISO	PRODUCT CODE			SIZE & LENGTH			ACTUAL LENGTH						
				Head Diameter	Head Length	Total Length	0	5	10	15	20	25	30
Diamond, Type, Shape													
806 314 197	CF-SS25	994664	S.Standard	1.5	6.5	15.5							
806 314 197	CF-S25	994657	Standard	1.5	6.5	17.5							
806 314 197	CF-25	994603	Standard	1.5	6.5	19.5							
806 314 299	CF-S26	994658	S.Standard	1.7	6.5	17							
806 314 299	CF-26	994604	Standard	1.7	6.5	19							
806 314 298	CF-S28	994659	Standard	1.8	8	19.5							
806 314 298	CF-28	994605	L.Standard	1.8	8	21.5							
806 314 298	CF-28C	994725	Coarse	1.8	8	21.5							
806 314 298	CF-28F	994772	Fine	1.8	8	21.5							
806 314 298	CF-L28	994673	Standard	1.8	8	23.5							
806 314 198	CF-S21	994656	S.Standard	1.1	8	17							
806 314 198	CF-21	994601	Standard	1.1	8	19							
806 314 198	CF-21C	994723	Coarse	1.1	8	19							
806 314 198	CF-21F	994770	Fine	1.1	8	19							
806 314 298	CF-14	994597	Standard	1.5	8.5	21.5							
806 314 298	CF-14C	994715	Coarse	1.5	8.5	21.5							
806 314 298	CF-14F	994768	Fine	1.5	8.5	21.5							
806 314 298	CF-31	994606	Standard	1.9	8.5	20							
806 314 298	CF-31C	994726	Coarse	1.9	8.5	20							
806 314 298	CF-31F	994773	Fine	1.9	8.5	20							
806 314 298	CF-L31	994674	Standard	1.9	8.5	22							
806 314 198	CF-11	994594	Standard	1.0	8.5	21.5							
806 314 198	CF-11C	994713	Coarse	1.0	8.5	21							
806 314 198	CF-11F	994766	Fine	1.0	8.5	21							
806 314 299	CF-22	994602	Standard	1.7	9.5	21							
806 314 299	CF-22C	994724	Coarse	1.7	9.5	21							
806 314 299	CF-22F	994771	Fine	1.7	9.5	21							
806 314 299	CF-22EF	994784	ExtraFine	1.7	9.5	21							
806 314 299	CF-32	994607	Standard	2.1	9.5	23							
806 314 299	CF-L32	994675	L.Standard	2.1	9.5	25							
806 314 298	CF-15	994598	Standard	1.7	10	21							
806 314 198	CF-12	994595	Standard	1.3	10	21							
806 314 198	CF-13	994596	Standard	1.4	10	22							
806 314 198	CF-13C	994714	Coarse	1.4	10	22							
806 314 198	CF-13F	994767	Fine	1.4	10	22							
806 314 298	CF-17	994599	Standard	1.6	10	22							
806 314 298	CF-17C	994722	Coarse	1.6	10	22							
806 314 298	CF-17F	994769	Fine	1.6	10	22							
806 314 198	CF-18	994600	Standard	1.9	12	24							
806 314 198	CF-19	994672	Standard	1.8	12	26							

BURS

Nu2medical

BURS

Diamond Bur - Straight Flat End

10pcs/pack

\$9.65
/pack

Coarse / 534 / 125-150 μ

Standard / 524 / 106-125 μ

Fine / 514 / 53-63 μ

ExtraFine / 504 / 20-30 μ

ISO	PRODUCT CODE			SIZE & LENGTH			ACTUAL LENGTH					
				Head Diameter	Head Length	Total Length	0	5	10	15	20	25
Diamond, Type, Shape												
806 314 107	SF-S32	994636	S.Standard	0.8	4	16.5						
806 314 107	SF-32	994542	Standard	0.8	4	19.5						
806 314 109	SF-S41	994637	S.Standard	1.0	4	19						
806 314 109	SF-41	994543	Standard	1.0	4	19						
806 314 109	SF-41C	994700	Coarse	1.2	4	19						
806 314 107	SF-S31	994635	S.Standard	1.3	4.5	17						
806 314 109	SF-31	994541	Standard	1.3	4.5	19						
806 314 111	SF-22	994540	Standard	1.4	6	19.5						
806 314 110	SF-21	994539	Standard	1.4	7	21						
806 314 110	SF-21C	994699	Coarse	1.4	7	21						
806 314 111	SF-S11	994633	Standard	1.2	8	20						
806 314 111	SF-11	994536	L.Standard	1.2	8	22						
806 314 111	SF-11C	994696	Coarse	1.2	8	22						
806 314 111	SF-S12	994634	Standard	1.4	8	19.5						
806 314 111	SF-12	994537	L.Standard	1.4	8	22						
806 314 111	SF-12C	994697	Coarse	1.4	8	22						
806 314 111	SF-L12	994671	Standard	1.4	8	24						
806 314 111	SF-13	994538	Standard	1.6	8	22						
806 314 111	SF-13C	994698	Coarse	1.6	8	22						

Diamond Bur - Straight Ogi-val End

10pcs/pack

\$9.65
/pack

Coarse / 534 / 125-150 μ

Standard / 524 / 106-125 μ

Fine / 514 / 53-63 μ

ExtraFine / 504 / 20-30 μ

ISO	PRODUCT CODE			SIZE & LENGTH			ACTUAL LENGTH					
				Head Diameter	Head Length	Total Length	0	5	10	15	20	25
Diamond, Type, Shape												
806 314 288	SO-SS20	994662	Standard	1.2	6	16.5						
806 314 288	SO-S20	994638	Standard	1.2	6	18.5						
806 314 288	SO-20	994545	Standard	1.2	6	21						
806 314 130	SO-24	994547	Standard	1.2	6	19						
806 314 289	SO-21	994546	Standard	1.4	8	21.5						
806 314 131	SO-15	994544	Standard	1.4	10	22						

Diamond Bur - Rounded Shoulder

10pcs/pack

\$9.65
/pack

Coarse / 534 / 125-150 μ

Standard / 524 / 106-125 μ

Fine / 514 / 53-63 μ

ExtraFine / 504 / 20-30 μ

ISO	PRODUCT CODE			SIZE & LENGTH			ACTUAL LENGTH					
				Head Diameter	Head Length	Total Length	0	5	10	15	20	25
Diamond, Type, Shape												
806 314 544	RS-31	994556	Standard	1.8	4.5	19						
806 314 545	RS-21	994555	Standard	1.8	7.5	21						
806 314 553	RS-11	994554	Standard	1.9	10.5	23						

BURS

NuMedical

BURS

Diamond Bur - Straight Round End
10pcs/pack

\$9.65 /pack

Coarse / 534 / 125-150 μ
Standard / 524 / 106-125 μ
Fine / 514 / 53-63 μ
ExtraFine / 504 / 20-30 μ

ISO	PRODUCT CODE			SIZE & LENGTH			ACTUAL LENGTH					
				Head Diameter	Head Length	Total Length	0	5	10	15	20	25
Diamond, Type, Shape												
806 314 140	SR-18	994552	Standard	1.2	6	21.5						
806 314 140	SR-18C	994720	Coarse	1.2	6	21.5						
806 314 140	SR-18F	994751	Fine	1.2	6	21.5						
806 314 140	SR-19	994553	Standard	1.4	6	22						
806 314 140	SR-19C	994721	Coarse	1.4	6	22						
806 314 141	SR-10	994548	Standard	1.0	8	22						
806 314 141	SR-10F	994746	Fine	1.0	8	22						
806 314 141	SR-11	994549	Standard	1.2	8	22						
806 314 141	SR-11C	994716	Coarse	1.2	8.1	22						
806 314 141	SR-11F	994747	Fine	1.2	8	22						
806 314 141	SR-11EF	994797	ExtraFine	1.2	8	22						
806 314 141	SR-13	994551	Standard	1.6	8	22						
806 314 141	SR-13C	994718	Coarse	1.6	8	22						
806 314 141	SR-13F	994749	Fine	1.6	8	22						
806 314 141	SR-14C	994719	Coarse	1.8	8	22						
806 314 141	SR-14F	994750	Fine	1.8	8	22						
806 314 141	SR-12	994550	Standard	1.4	8.5	22						
806 314 141	SR-12C	994717	Coarse	1.4	8.5	22						
806 314 141	SR-12F	994748	Fine	1.4	8.5	22						
806 314 141	SR-12EF	994798	ExtraFine	1.4	8.5	22						

Diamond Bur - Marginal Shaper
10pcs/pack

\$9.65 /pack

Coarse / 534 / 125-150 μ
Standard / 524 / 106-125 μ
Fine / 514 / 53-63 μ
ExtraFine / 504 / 20-30 μ

ISO	PRODUCT CODE			SIZE & LENGTH			ACTUAL LENGTH					
				Head Diameter	Head Length	Total Length	0	5	10	15	20	25
Diamond, Type, Shape												
806 314 299	MS-13	994514	Standard	1.6	8	19.5						
806 314 299	MS-14	994515	Standard	1.9	9	22.5						

Diamond Bur - CR Type
10pcs/pack

\$9.65 /pack

Coarse / 534 / 125-150 μ
Standard / 524 / 106-125 μ
Fine / 514 / 53-63 μ
ExtraFine / 504 / 20-30 μ

ISO	PRODUCT CODE			SIZE & LENGTH			ACTUAL LENGTH					
				Head Diameter	Head Length	Total Length	0	5	10	15	20	25
Diamond, Type, Shape												
806 314 196	CR-12F	994753	Fine	1.5	4	19						
806 314 196	CR-12EF	994796	ExtraFine	1.4	4	19						
806 314 196	CR-11F	994752	Fine	2.0	6	20						
806 314 196	CR-11EF	994795	ExtraFine	1.9	6	20						
806 314 197	CR-21F	994754	Fine	1.2	7	21.5						
806 314 289	CR-22F	994755	Fine	1.3	8	21.5						

BURS

Nu2medical

BURS

Diamond Bur - Taper Cone
Tip
10pcs/pack

\$9.65
/pack

Coarse / 534 / 125-150 μ

Standard / 524 / 106-125 μ

Fine / 514 / 53-63 μ

ExtraFine / 504 / 20-30 μ

ISO	PRODUCT CODE			SIZE & LENGTH			ACTUAL LENGTH					
				Head Diameter	Head Length	Total Length	0	5	10	15	20	25
Diamond, Type, Shape												
806 314 160	TC-26	994520	Standard	1.2	6	21						
806 314 166	TC-S20	994629	S.Standard	1.1	8	17						
806 314 166	TC-20	994518	Standard	1.1	8	20						
806 314 160	TC-S21	994630	Standard	1.4	8	20						
806 314 160	TC-S21C	994728	Coarse	1.4	8	20						
806 314 160	TC-21	994519	L.Standard	1.4	8	22						
806 314 160	TC-21F	994737	Fine	1.1	8	22						
806 314 160	TC-21EF	994783	ExtraFine	1.4	8	22						
806 314 166	TC-10	994516	Standard	1.1	10	22						
806 314 160	TC-S11	994628	Standard	1.6	10	20						
806 314 160	TC-11	994517	L.Standard	1.6	10	22						
806 314 160	TC-11C	994690	Coarse	1.6	10	22						
806 314 160	TC-11F	994736	Fine	1.6	10	22						
806 314 160	TC-11EF	994782	ExtraFine	1.6	10	22						
806 314 166	TC-L19	994669	Standard	1.9	13	25						

BURS

Diamond Bur - CD Type
10pcs/pack

\$9.65
/pack

Coarse / 534 / 125-150 μ

Standard / 524 / 106-125 μ

Fine / 514 / 53-63 μ

ExtraFine / 504 / 20-30 μ

ISO	PRODUCT CODE			SIZE & LENGTH			ACTUAL LENGTH					
				Head Diameter	Head Length	Total Length	0	5	10	15	20	25
Diamond, Type, Shape												
806 314 001	CD-50F	994756	Fine	0.6	0.6	17						
806 314 254	CD-55F	994761	Fine	0.9	1.2	17						
806 314 138	CD-52F	994757	Fine	0.7	3.5	17						
806 314 161	CD-53F	994759	Fine	0.7	2.5	16.5						
806 314 246	CD-54F	994760	Fine	0.6	2.5	16.5						
806 314 246	CD-56F	994762	Fine	0.7	2.5	16.5						
806 314 138	CD-51F	994758	Fine	0.7	2.3	17						
806 314 108	CD-58F	994764	Fine	0.8	3.5	16.5						
806 314 247	CD-57F	994763	Fine	0.7	4	16.5						
806 314 171	CD-59F	994765	Fine	1.3	6.5	16.5						

Nu
medical

BURS

Diamond Bur - Flame Ogival
End
10pcs/pack

\$9.65
/pack

Coarse / 534 / 125-150 μ

Standard / 524 / 106-125 μ

Fine / 514 / 53-63 μ

ExtraFine / 504 / 20-30 μ



ISO	PRODUCT CODE			SIZE & LENGTH			ACTUAL LENGTH					
				Head Diameter	Head Length	Total Length	0	5	10	15	20	25
Diamond, Type, Shape												
806 314 257	FO-30	994533	Standard	1.8	3	19						
806 314 257	FO-30C	994693	Coarse	1.8	3	19						
806 314 257	FO-30F	994740	Fine	1.8	3.5	19						
806 314 257	FO-32C	994694	Coarse	1.8	3.5	19.5						
806 314 257	FO-32	994534	Standard	1.6	4	19						
806 314 540	FO-38EF	994789	ExtraFine	0.7	4	22						
806 314 540	FO-38	994535	Standard	0.9	4	22						
806 314 257	FO-29	994532	Standard	1.9	4.5	19.5						
806 314 257	FO-29EF	994787	ExtraFine	1.9	4.5	19.5						
806 314 243	FO-40EF	994790	ExtraFine	1.0	4.5	23.5						
806 314 257	FO-25	994530	Standard	2.8	5.1	19						
806 314 257	FO-25C	994691	Coarse	2.8	5.0	20						
806 314 257	FO-27	994531	Standard	3.2	5	19						
806 314 257	FO-28C	994692	Coarse	2.1	5	19.5						
806 314 248	FO-20EF	994785	ExtraFine	1.4	6	20						
806 314 248	FO-41EF	994791	ExtraFine	1.2	6.5	19						
806 314 249	FO-S15	994631	Standard	1.1	7	19.5						
806 314 249	FO-15	994525	Standard	1.1	7	21.5						
806 314 298	FO-S21	994632	Standard	1.4	7	18						
806 314 298	FO-21	994527	Standard	1.4	7	21						
806 314 298	FO-21F	994738	Fine	1.4	7	21						
806 314 298	FO-21EF	994786	ExtraFine	1.4	7	21						
806 314 298	FO-22	994528	Standard	1.6	7	21						
806 314 298	FO-22F	994739	Fine	1.6	7	21						
806 314 298	FO-22EF	994788	ExtraFine	1.6	7	21						
806 314 249	FO-16	994526	Standard	1.3	8	21						
806 314 249	FO-23	994529	Standard	1.6	8	21						
806 314 248	FO-42EF	994792	ExtraFine	1.2	8	22						
806 314 190	FO-54C	994695	Coarse	1.8	9.5	21						
806 314 299	FO-11	994521	Standard	1.2	10	24						
806 314 250	FO-12	994522	Standard	1.4	10	22						
806 314 250	FO-13	994523	Standard	1.7	10	22						
806 314 251	FO-14	994524	Standard	2.3	10	22						

BURS

NuMedical

Diamond Bur - Wheel Round Edge 10pcs/pack

**\$9.65
/pack**

Coarse / 534 / 125-150 μ

Standard / 524 / 106-125 μ

Fine / 514 / 53-63 μ

ExtraFine / 504 / 20-30 μ

ISO	PRODUCT CODE			SIZE & LENGTH			ACTUAL LENGTH						
				Head Diameter	Head Length	Total Length	0	5	10	15	20	25	
806 314 304	WR-S31F	994774	Fine	1.6	0.6	16.4							
806 314 304	WR-31F	994745	Fine	1.6	0.6	19							
806 314 068	WR-11	994589	Standard	4.2	0.9	19							
806 314 068	WR-18	994592	Standard	4.2	1	19							
806 314 068	WR-12C	994710	Coarse	4.0	1.6	19							
806 314 068	WR-12	994590	Standard	4.2	1.6	19							
806 314 068	WR-13C	994711	Coarse	4.2	2	19							
806 314 068	WR-13	994591	Standard	4.2	2	19							
806 314 903	WR-19C	994712	Coarse	4.3	2.2	19.5							
806 314 903	WR-19	994593	Standard	4.3	2.2	19.5							

BURS

Diamond Bur - Ball Round Type 10pcs/pack

**\$9.65
/pack**

Coarse / 534 / 125-150 μ

Standard / 524 / 106-125 μ

Fine / 514 / 53-63 μ

ExtraFine / 504 / 20-30 μ

ISO	PRODUCT CODE			SIZE & LENGTH			ACTUAL LENGTH						
				Head Diameter	Head Length	Total Length	0	5	10	15	20	25	
806 314 001	BR-48F	994744	Fine	0.6	0.6	19							
806 314 001	BR-S49	994644	Standard	0.8	0.8	17							
806 314 001	BR-49	994575	Standard	0.8	0.8	19							
806 314 001	BR-S45	994642	Standard	1.0	1.0	16.5							
806 314 001	BR-45	994573	Standard	1.0	1.0	19							
806 314 001	BR-S46	994643	Standard	1.2	1.2	17							
806 314 001	BR-46	994574	Standard	1.2	1.2	19							
806 314 001	BR-S41	994641	Standard	1.4	1.4	17							
806 314 001	BR-41	994572	Standard	1.4	1.4	19							
806 314 001	BR-40	994571	Standard	1.6	1.6	19							
806 314 001	BR-40EF	994794	ExtraFine	1.4	1.4	19							
806 314 001	BR-S31	994640	Standard	1.8	1.8	16.5							
806 314 001	BR-31	994570	Standard	1.8	1.8	19							
806 314 001	BR-31C	994706	Coarse	1.8	1.8	19							
806 314 001	BR-L31	994670	Standard	1.8	1.8	22							
806 314 001	BR-30C	994705	Coarse	2.3	2.3	19							
806 314 001	BR-30	994569	Standard	1.2	2.3	19							

NuMedical

BURS

Diamond Bur - Ball Collar
Type
10pcs/pack

\$9.65
/pack

Coarse / 534 / 125-150 μ

Standard / 524 / 106-125 μ

Fine / 514 / 53-63 μ

ExtraFine / 504 / 20-30 μ

ISO	PRODUCT CODE			SIZE & LENGTH			ACTUAL LENGTH						
				Head Diameter	Head Length	Total Length	0	5	10	15	20	25	
806 314 002	BC-31	994576	Standard	1.8	2.5	19							
806 314 002	BC-S42	994647	Standard	1.4	2.5	16.5							
806 314 002	BC-42	994577	Standard	1.4	2.5	19							
806 314 002	BC-S43	994648	Standard	1.2	3	16.5							
806 314 002	BC-43	994578	Standard	1.2	3	19							
806 314 002	BC-S44	994645	Standard	1.2	3	17							
806 314 002	BC-44	994579	Standard	1.0	3	19							
806 314 002	BC-S45	994646	S.Standard	1.0	3	17							
806 314 002	BC-45	994580	Standard	1.2	3	19							

Diamond Bur - Single Invert-
ed Cone
10pcs/pack

\$9.65
/pack

Coarse / 534 / 125-150 μ

Standard / 524 / 106-125 μ

Fine / 514 / 53-63 μ

ExtraFine / 504 / 20-30 μ

ISO	PRODUCT CODE			SIZE & LENGTH			ACTUAL LENGTH						
				Head Diameter	Head Length	Total Length	0	5	10	15	20	25	
806 314 010	SI-S45	994652	Standard	1.0	1.6	16.5							
806 314 010	SI-45	994584	Standard	1.0	1.5	19.5							
806 314 010	SI-S46	994653	Standard	1.2	1.5	16.5							
806 314 010	SI-46	994585	Standard	1.2	1.6	19.5							
806 314 010	SI-S47	994654	Standard	1.4	1.8	17							
806 314 010	SI-47C	994707	Coarse	1.8	1.8	19							
806 314 010	SI-47	994586	Standard	1.4	1.8	19							
806 314 010	SI-S48	994655	Standard	1.6	1.8	16.5							
806 314 010	SI-48C	994708	Coarse	1.6	1.8	19							
806 314 010	SI-48	994587	Standard	1.6	1.8	19							
806 314 010	SI-50C	994709	Coarse	2.3	2.5	19.5							
806 314 010	SI-50	994588	Standard	2.3	2.5	19.5							

BURS

NuMedical

BURS

Diamond Bur - Double Inverted Cone
10pcs/pack

\$9.65 /pack

Coarse / 534 / 125-150 μ

Standard / 524 / 106-125 μ

Fine / 514 / 53-63 μ

ExtraFine / 504 / 20-30 μ

ISO	PRODUCT CODE			SIZE & LENGTH			ACTUAL LENGTH					
				Head Diameter	Head Length	Total Length	0	5	10	15	20	25
806 314 019	DI-SS41	994663	Standard	1.0	2.3	14.5						
806 314 019	DI-S41	994650	Standard	1.0	2.3	17						
806 314 019	DI-41	994582	Standard	1.0	2.3	19						
806 314 019	DI-S40	994649	Standard	1.0	3	17.5						
806 314 019	DI-40	994581	Standard	1.0	3	19.5						
806 314 019	DI-S42	994651	Standard	1.4	3.3	17						
806 314 019	DI-42	994583	Standard	1.4	3.3	19						

Diamond Bur - Extra Special Shape
10pcs/pack

\$9.65 /pack

Coarse / 534 / 125-150 μ

Standard / 524 / 106-125 μ

Fine / 514 / 53-63 μ

ExtraFine / 504 / 20-30 μ

ISO	PRODUCT CODE			SIZE & LENGTH			ACTUAL LENGTH					
				Head Diameter	Head Length	Total Length	0	5	10	15	20	25
806 314 150	EX-17F	994741	Fine	1.0	0.4	21						
806 314 150	EX-18F	994742	Fine	1.2	0.4	21						
806 314 237	EX-41	994568	Standard	1.0	3	19						
806 314 234	EX-31	994567	Standard	1.4	4.5	19						
806 314 038	EX-14C	994702	Coarse	4.3	4.5	19.5						
806 314 039	EX-11C	994701	Coarse	3.2	5	19						
806 314 039	EX-11	994557	Standard	3.2	5	19						
806 314 237	EX-S20	994639	Standard	1.8	5	17.5						
806 314 237	EX-20	994559	Standard	1.8	5	19						
806 314 237	EX-21	994560	Standard	1.2	5	19						
806 314 237	EX-21F	994743	Fine	1.2	5.5	20						
806 314 237	EX-21EF	994793	ExtraFine	1.2	5	20						
806 314 237	EX-26C	994704	Coarse	3.2	5	19						
806 314 237	EX-26	994563	Standard	3.2	5	19						
806 314 655	EX-27	994564	Standard	1.6	7	20.5						
806 314 039	EX-12	994558	Standard	3.5	7	20						
806 314 002	EX-23	994561	Standard	1.8	7.5	19.5						
806 314 655	EX-28	994565	Standard	1.8	8	21.5						
806 314 234	EX-25C	994703	Coarse	2.1	8	20						
806 314 655	EX-29	994566	Standard	2.1	9	22.5						
806 314 220	EX-24	994562	Standard	1.7	10	24						

BURS

NuMedical